# EN Lundatex® sock

A compression stocking providing safe and precise pressure at all times

The unique medical Lundatex® sock provides safe, comfortable pressure at all times, whether at rest or when active, regardless of foot and leg shape. The product ensures a precise pressure at all times, even after oedema reduction.

## INDICATIONS:

- |                        |                            |             |
|------------------------|----------------------------|-------------|
| - Venous insufficiency | - Mixed leg ulcers         | - Burns     |
| - Oedema               | - Varicose veins           | - Pregnancy |
| - Lymphoedema          | - Post-thrombotic syndrome | - Flights   |
| - Venous leg ulcers    | - Deep vein thrombosis     |             |

⚠ Consult a medical doctor before using compression products if you have:

- Arterial circulatory disturbances
- Reduced arterial pressure (ABPI <0.8)
- Diabetes
- Heart disease

**Important:** Remove the product at night, and/or if discomfort or pain occurs.

The sock is re-useable and washable. See the packaging for washing and care instructions. Avoid using oily skin lotions. Wash the sock after usage of lotions.

Lundatex® sock can be used in the following ways:

1. On its own, when compression is only necessary on the foot and ankle region;
2. Together with Lundatex® medical and/or Lundatex® system, **instead of** bandage wrapping on the foot, to prevent bandage slippage;
3. Together with Lundatex® medical and/or Lundatex® system, **underneath** the bandage on the foot, to achieve a greater foot compression.

MiniLock® is a fastening device that secures the bandage end to the cuff of the sock.

## INSTRUCTIONS FOR USE:

### Sizing

1. Check your normal shoe size to determine whether you are within size S/M or M/L.

### Applying the sock

2. Grab the cuff using both hands, and pull the sock over the foot and heel in one single motion.

3. Use your hands to adjust and straighten out any creases or irregularities.

### For usage together with Lundatex® medical / Lundatex® system


1. Attach the MiniLock® to the sock by placing it on the back of your ankle, below the differently-coloured cuff, with the opening facing the inside of your leg.

2. Open the MiniLock®, place the end of the Lundatex® medical bandage inside, and close it.

3. Start wrapping the bandage from the inside of the ankle up and along the leg, in circular turns, according to the IFU for Lundatex® medical.

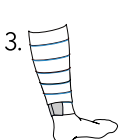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